

# CoaguChek XS Pro/Pro II

## Docking/Undocking Device



### Docking the CoaguChek

1. Place the CoaguChek Pro II meter on the handheld base unit (HBU), pushing it gently towards the gold charging contacts, lower to engage the location pins and lock into position.



2. When the device is correctly docked on the HBU and all cables are connected properly, battery charging and data transfer will start automatically.

3. These two icons will appear on the CoaguChek Pro II screen when the meter is correctly docked into the HBU.

| Button/Icon   | Meaning  |
|---|--|
|  | Communication is taking place via the infrared interface |
|  | Operation with power supply adapter                      |

### Undocking the CoaguChek

4. When removing the CoaguChek Pro II meter from the HBU, gently push it forward, then slightly angle up and pull out of the HBU.

