

Important Notice

When i-STAT cartridges are removed from the refrigerator and stored on the bench, they must be labelled with their new expiry date. e.g:

- Troponin - 14 days from removal
- Chem 8 - 14 days from removal
- CG4 - 2 months from removal
- Creatinine - 14 days from removal
- Glucose - 14 days from removal
- CG8 - 2 months from removal

For sites that take out whole boxes at a time, the “expiry date” MUST be recorded on the box packaging.



NO CARTRIDGES ARE TO BE TRANSFERRED BETWEEN BOXES