

epoc Maintenance Log

Location: _____ Year: _____

Instrument exterior and touchscreen should be cleaned and ready for use each day or as required.

Cleaning Instructions

Touch screen: Clean the touchscreen using a soft cloth dampened slightly with water

- Exterior:**
1. Dampen a soft cleaning cloth with water or mild cleaning solution (be sure that the cleaning cloth is only moist, not dripping wet).
 2. Carefully wipe the face of the touch screen free of fingerprints and other smudges.
 3. Remove any blood or dust for the outer surface of the instrument case using a clean, soft cloth moistened with 10% bleach.

Monthly(sign/date)		Jan	Feb	Mar	Apr	May	Jun	Jul	Aug	Sep	Oct	Nov	Dec
Run QC	GAS-ISE Level 1												
	GAS-ISE Level 2												
	Hct Level 1												
	Hct Level 2												
Order received, accepted & Picking Slip completed.													
Run QAP													
epoc Thermal QA run													

Issue Log (please provide date and description of problem & solution)
