

# CoaguChek XS Pro/Pro II

## Cleaning Instructions

Please follow the procedures below to clean and disinfect the CoaguChek XS Pro/Pro II. Failure to follow the procedures may cause malfunction of the device.

### Recommended cleaning/disinfecting agents

- A soft cloth slightly dampened (not wet) with a small amount of liquid soap diluted in water
- Rubbing alcohol (70% ethanol or isopropyl alcohol)
- Alcohol-based disinfectant (a mixture of 1-propanol (400mg/g), 2-propanol (200 mg/g) and glutaraldehyde (1.0 mg/g))
- 10% sodium hypochlorite solution (1 part bleach to 9 parts de-ionized water, made fresh every 24 hours)
- Disposable wipes containing quaternary ammonium compounds up to 0.5% (single compound or mixture) in isopropyl alcohol (isopropanol) up to 55%



1. Wear gloves while cleaning/disinfecting
2. Use one of the cleaning solutions recommended above

### Cleaning/disinfecting the exterior (meter housing)

1. Ensure that the blue test strip guide cover remains tightly closed while cleaning the device housing. Do not let liquid accumulate near any opening and ensure that no liquid enters the device. Using a soft, lint-free cloth slightly dampened (not wet), wipe away residual moisture and fluids after cleaning the housing
2. Allow wiped area to **dry for at least 10 minutes** before performing the next patient test

### Cleaning/disinfecting the test strip guide & cover

1. With the device powered off, remove the test strip guide cover to clean it. (Use your thumbnail to open the cover of the test strip guide by pressing its front edge upward.) Move the cover safely away from the device. Then rinse the cover with warm water or wipe it clean using the solutions recommended above. Let the test strip guide cover dry **for at least 10 minutes** before re-attaching it.
2. Hold the device upright with test strip guide facing down
3. With a soft, lint-free cloth slightly dampened (not wet), apply agent for the minimum required contact time (refer to the cleaning/disinfecting agent's labelling; e.g. 30 sec).
4. Wipe away residual moisture and fluids
5. Allow the inside of the test strip guide to **dry for a least 10 minutes** before re-attaching the test strip guide cover and start testing again



**Please contact your local PoCT Coordinator if the following error occurs after cleaning:** E-602 General System Error. The device is defective. No test is possible.



*Useful tips on how to minimize the risk of contaminating the device with blood: apply blood via side-dosing directly from the fingertip. If pipettes or syringes are used, the blood volume should not exceed 10µL*